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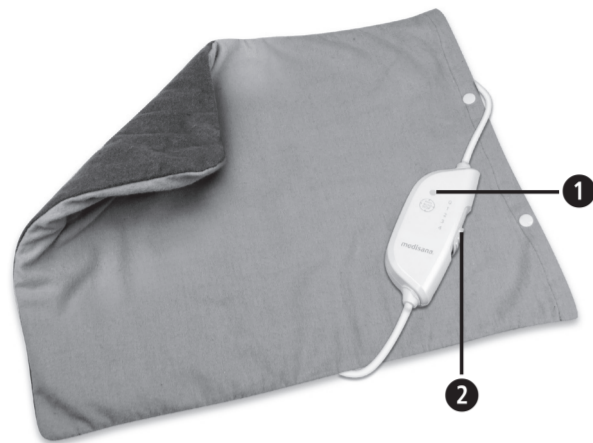
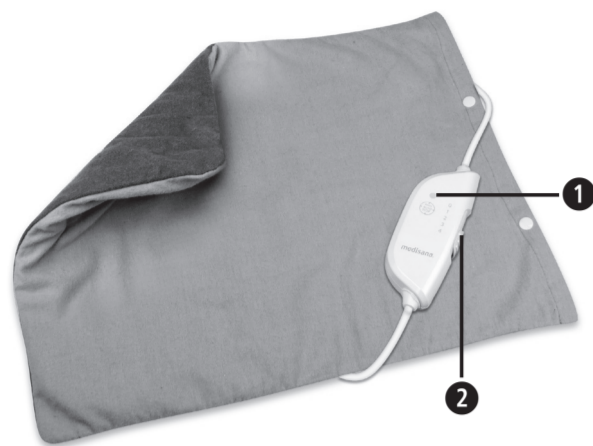
- 1 Power LED
- 2 Slider control

### Explanation of symbols

Do not use the heated cushion when it is folded!

Do not puncture the heated cushion

Not suitable for children under 3 years!

Only use the heated cushion indoors!

You may wash the cover in a normal wash at max. 30 °C. Follow the notes the label!

The heating pad can be washed by hand!

Do not bleach!

Do not dry the heated cushion in a tumble drier!

Do not iron the heated cushion!

Do not dry clean!

**IMPORTANT!**  
Non-observance of these instructions can result in serious injury or damage to the device.

**WARNING**  
These warning notes must be observed to prevent any injury to the user.

**CAUTION**  
These notes must be observed to prevent any damage to the device..

**NOTE**  
These notes provide useful additional information on the installation or operation.

Protection category II

**LOT** LOT number

Manufacturer

## EN Safety instructions

**Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.**

### Power supply

- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- The control switch should not be covered or placed on or below the pad while the unit is in operation.
- Keep the mains lead away from hot surfaces.
- Never carry, pull or turn the heat pad by the mains lead and do not let the lead get tangled.
- Never touch a device that has fallen into water. Unplug the unit from the mains outlet immediately.
- The controls and leads must not be exposed to any kind of moisture.
- The heat pad may only be operated with the control unit (MAG 01HP) supplied.

### For special people

- Do not use the heat pad on children or persons who are disabled, asleep or insensitive to heat (persons who cannot react on overheating).
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Children below the age of 3 years are not allowed to use this device, because they are not able to react on overheating.
- The heat pad must not be used by young children over the age of 3 years, unless the control unit has been adjusted accordingly by its parents or legal guardians or unless the child has been well instructed how to use the control unit properly.
- Should you have any concerns about health issues, consult your doctor before using the heat pad.
- Do not use the heat pad on parts of the body that are swollen, injured or inflamed.
- Should you experience persistent pains in the muscles and joints, please inform your doctor. Persistent pains can be a symptom of serious illness.
- If you wear a pacemaker, please be aware that the electromagnetic fields emanating from this electric product may, under certain circumstances, interfere with the functions of your pacemaker. Therefore, ask your doctor and the manufacturer of the pacemaker for advice before using this product.
- If you experience any pain or discomfort while using the unit, stop using it immediately.

### Operating the device

- Only use the device for its intended use as in the instruction manual.
- If misused, all guarantee rights shall become null and void.
- Carefully check the heat pad before each use, if you notice wear or damage.
- Do not use if you notice wear, damage or signs of improper use on the heat pad, the switch or the cables or if the device is not working.
- Do not fold or crease the heat pad during use.

- Neither pierce the pad with, nor attach safety pins or other spiky or sharp objects to the pad.
- Never use the pad unsupervised.
- The device is only for private use and is not to be used in hospitals.
- Do not fall asleep while the heat pad is switched on. Prolonged use of the heated cushion at a high setting could lead to skin burns.
- Do not cover the pad with any other cushions.
- Do not sit on the heat pad. Instead, rest the pad on or against the part of the body to be treated.
- Do not use the heat pad when wet and only use it in dry environments (not in the bathroom or similar environments).

### Maintenance and cleaning

- You are only permitted to clean the unit. To avoid hazards, never repair the device yourself. Contact the customer service.
- Cleaning and user maintenance shall not be made by children without supervision.
- If the power supply cable is damaged, it may only be replaced by medisana, an authorised dealer or by qualified personnel in order to avoid hazards.
- A damaged power supply cable must be replaced with a power cable of the same type.
- If a fault occurs, do not attempt to repair the unit yourself. Repairs must only be carried out by an authorized specialist dealer or other suitably qualified personnel.
- When storing the heat pad, allow it to cool down before laying out flat in a clean and dry place.
- Do not store the device with any objects covering or resting on top of it in order to avoid sharp crinkles.

### Scope of supply

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use the it and contact your dealer or your service centre.

The following parts are included:

- 1 **medisana** Heat pad **HP 605** with removable switching unit
- 1 Instruction manual

If you notice any transport damage during unpacking, please contact your dealer without delay.

**WARNING**  
**Please ensure that the polythene packing is kept away from the reach of children!**  
**Risk of suffocation!**

### Operating

Make sure, the connection cable of the control unit is properly connected to the plug of the heat pad. Plug the unit into the mains outlet and move the slider control 2 from position 0 to position 1. The power LED 1 will light up in green.

After a few minutes you will be able to feel the pad getting noticeably warmer. If you require a higher temperature setting, move the switch to the next positions 2 or 3 or the maximum heat setting, position 4. If the heat pad gets too hot for you, move the slider back to position 3, 2 or 1. After a short time, the heat pad will cool down.

The heat pad may cool down after a short period of time. This is normal. The fast re-heating according to the setting will follow immediately.

To switch the unit off, move the switch back to position 0. The power LED 1 will go out, indicating that the unit is switched off. Unplug the unit from the mains outlet.

Move the switch to position 1 for any continuous use. The heat pad will switch off automatically after approx. 90 minutes of continuous use.

To switch it back on again, move the slider switch to position 0 and then back to the desired heat setting.

Unplug the unit from the mains outlet once you have finished using it.

### Troubleshooting

If there is no function at all after switching on the heat pad (power LED 1 shows no light), please check if the mains plug is correctly plugged in the mains outlet. If this is the case, the heat pad is defect. Please do not use the heat pad and contact the service center.

### Cleaning and maintenance

#### Heat pad cover

- Remove the heat pad from its cover.
- Dry clean the removable cover or as detailed in the instructions on the label.

#### Heat pad

- Before cleaning the pad, unplug it from the mains outlet and let it cool down for at least ten minutes.
- The heat pad is equipped with a removable control unit (MAG 01HP).
- Unplug the control unit from the heat pad by pulling out the connection cable from the plug on the heat pad.
- The heat pad can be cleaned dry (with a soft brush) or as described in the instructions on the product label.
- Never use strong detergents or hard brushes.
- To dry, lay the pad flat on a moisture-absorbent towel or mat.
- Only use the blanket again once it is completely dry.
- Reconnect the control unit to the heat pad in the correct manner.
- You should untangle the cable from time to time.
- Store the unit in a clean and dry place, laid out flat with nothing covering or resting on top of it.

### Disposal

This product must not be disposed of together with domestic waste.  
 All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

## Technical specifications

Name and model:	<b>medisana</b> Heat pad <b>HP 605</b>
Power supply:	220-240V~, 50 Hz
Heat output:	100 watts
Automatic switch-off:	after approx. 90 minutes
Dimensions approx.:	33 x 40 cm
Weight approx.:	480 g
Operating conditions:	Only use in dry rooms according to this user manual
Storage conditions:	Laid out flat, clean and dry
Art. number:	61148
EAN number:	40 15588 61148 3

**In accordance with our policy of continual product improvement, we reserve the right for technical and design changes without notice.**

The current version of this instruction manual can be found under [www.medisana.com](http://www.medisana.com)

## Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **medisana** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

**medisana GmbH**  
Carl-Schurz-Str. 2  
41460 NEUSS  
GERMANY

The service centre address is shown on the attached leaflet.

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